
Instructions for use of Mat Care Ovulation test Strip

Read these instructions carefully before performing the test

- Verify the package reads "Ovulation / LH".
- Remove the test device from the sealed pouch.
- Collect urine sample in a clean, dry container.
- Immerse the strip into the urine with the arrow end pointing towards the urine. Do not immerse past the MARK line. Take the strip out after a minimum of 5 seconds.
- Make sure the test has absorbed enough urine.
- Place the test on a flat surface.
- You can read results after 10 minutes: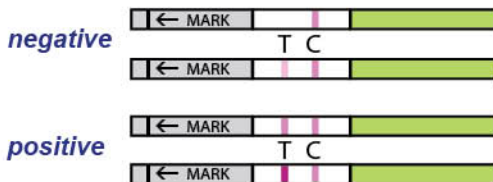

The T stands for Test band and the C stands for Control band.

Negative: When the test band does not appear or is less intense than the control band, the result is negative. Your LH surge has not yet started.

Positive: When the test band is of equal or greater intensity than the control band, the result is positive and you're ovulation takes place in 20 - 48 hours.

Invalid: When no bands appear, the test result is invalid. Repeat with a new test device and follow instructions carefully.

NOTE: do not interpret results after 20 minutes! Do not interpret results before the control band has appeared.

How does this test function?

The test detects LH (Luteinizing Hormone) in urine. The primary function of LH is releasing a mature egg from the ovaries.

The rapid increase of LH concentrate in urine, LH Surge, signals the start of the ovulation process. LH is always present in urine, but during the LH surge the level is 10-20 times higher than the regular level.

The LH surge lasts 1 - 3 days, after this period LH levels rapidly decreases to the regular level. The ovulation usually takes place in 20 - 48 hours after the start of the LH surge. This is the most fertile period of your menstrual cycle.

When should you start testing? Calculate the average length of your menstrual cycle. When you have an irregular cycle, base your calculations on your shortest cycle. Find your average in the table below. The right column shows the day on which you should start testing. The first day of your period, is considered day 1.

Average cycle	Start on day
25	7
26	8
27	9
28	10
29	11
30	12

Average cycle	Start on day
31	13
32	14
33	15
34	16
35	17
36	18

Notice the following when testing:

- Reduce liquid intake about 2 hours before collecting your urine as a diluted urine sample can prevent the test from detecting LH surge.
- Do NOT use morning urine for testing, but urine collected between 10:00 AM and 8:00 PM.
- Perform the test at about the same time everyday (preferably around 1:00 PM)
- Make sure the test absorbs sufficient urine.

How long should you continue testing? From the day you started testing, you perform a test every day until you have a positive result. This can last 5 to 15 days and this can vary by cycle.

What if no colored bands appear after 10 minutes? The test result is invalid. You have probably not used enough urine. Perform a new test and make sure you use enough urine.

Store and use at room temperature - Keep away from children - Keep away from direct sunlight, moisture and heat - This test is for single use only - Mat Care cannot be held liable for incorrect use of the test or incorrect interpretation of the test result - If you have any questions, please send an email to: info@matcare.com.

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